

STOCK CODE 603387

One Step Test for SARS-CoV-2 Antigen (Colloidal Gold) • For self-test

CE Marked


Rapid, Convenient, Easy and Reliable
Detection of COVID-19



Intended Use

One Step Test for SARS-CoV-2 Antigen (Colloidal Gold) is intended for the qualitative detection of SARS-CoV-2 antigens in human nasal swab samples.

This test is suitable for medical laypersons as a self-test at home or at work.

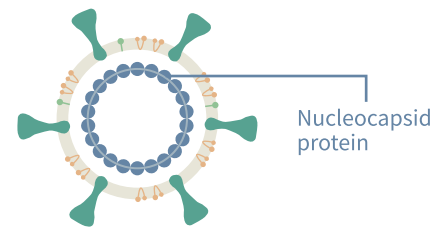

Product Components



SARS-CoV-2 antigen
test card




Biohazard sample bag



Extraction Tube with Sample
Extraction Solution and Tip




Sterile Swab

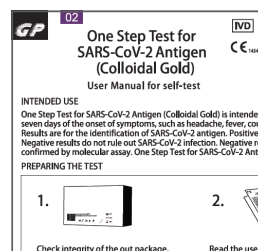

User manual

COVID-19 Diagnostic Testing

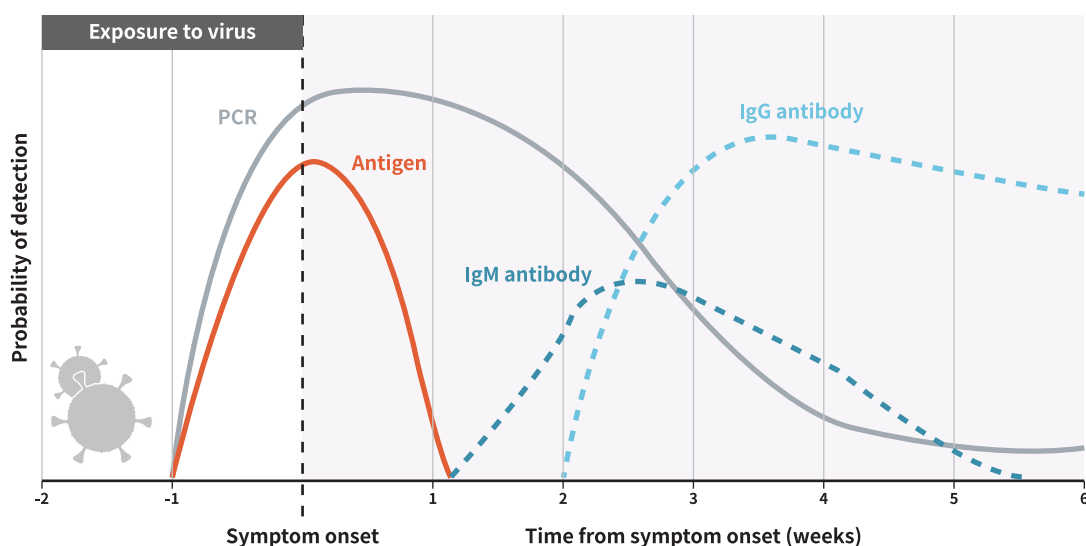

- **PCR-based tests** can detect small amounts of viral genetic materials.
- **Antigen tests** detect the presence of viral proteins and can return positive results when a person is most infectious.
- - - **Antibody tests** detect the body's immune response to the virus.

Features

-  Non-invasive sampling (Sample type: nasal swab)
-  Read test results visually. Do not require test equipment.
-  Early detection of SARS-CoV-2 infection
-  Rapid test. Test result available in 10-15 min.
-  Simple operation, easy to learn and use
-  Room temperature storage (4-30°C)

When to Use the Test Kit?

Use this test:


- ✓ If you want to test yourself.
- ✓ If you have symptoms similar to COVID-19, such as: E.g. headache, fever, cough, sore throat, loss of sense of smell or taste, shortness of breath, muscle pain.
- ✓ If you are concerned about whether you are infected with COVID-19.
- ✓ Use of the test by persons under 18 years of age only under the supervision of an adult.

Operation Video



Test Procedure

Nasal Swab Sampling >>

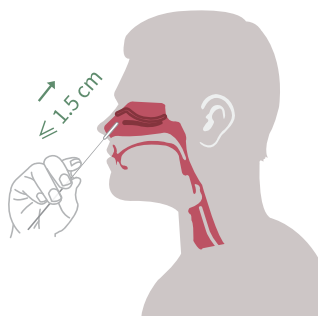

Insert the collection tip of swab into one nostril.




Rotate ≥ 4 times (15 s in total)




Repeat with the same swab.

Sample Pretreatment >>

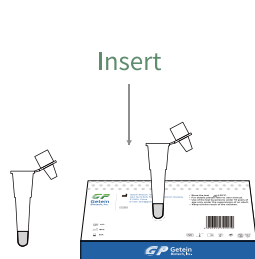

Remove the sealed film and insert it into the package.

Nasal swab
Rotate
10 times

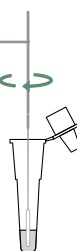


Squeeze
3 times



Tighten the disposable pipette

Test >>

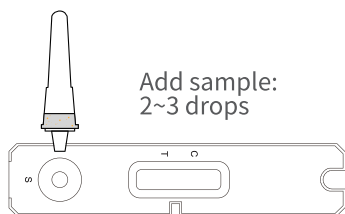


Step 1




Read result in 10-15 min

Step 2


Test Results

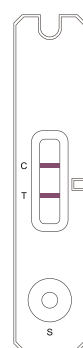


Positive

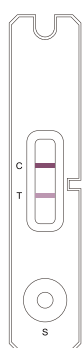


Positive

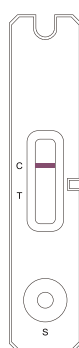

Negative

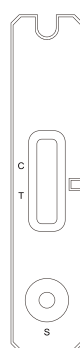

Invalid 1

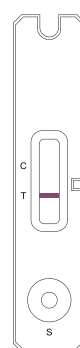

Invalid 2

Specifications

Product Name	One Step Test for SARS-CoV-2 Antigen (Colloidal Gold)
Test Item	SARS-CoV-2 Antigen
Package	1 T/kit, 5 T/kit, 7 T/kit, 25 T/kit
Product Code	CG20615/ CG206155/ CG206157/ CG2061525
Test Time	10-15 min
Storage Condition	4-30°C
Shelf Life	24 months
Recommended Test Temperature	23-25°C

Application Scenarios



Home



School



Work Place



Dormitory



Nursing Home



Cruise ship



Airport



Theater



Getein Biotech, Inc.

Add: No.9 Bofu Road, Luhe District, Nanjing, 211505, China

Tel: +86-25-68568508/68568594

Fax: +86-25-68568500

E-mail: sales@getein.com.cn; overseas@getein.com.cn

Web: en.bio-gp.com.cn



CMC Medical Devices & Drugs S.L.

Add: C/ Horacio Lengo N° 18, CP 29006, Málaga, Spain

Tel: +34951214054

